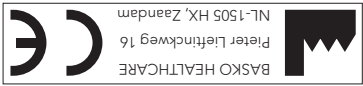
Basko Healthcare

Pieter Liefthickweg 16
NL-1505 HX Zaandam
Phone: +31 (0) 75 - 613 15 13
Fax: +31 (0) 75 - 612 63 73
E-mail: sales@basko.com
Internet: www.basko.com

Basko
Healthcare

Clubycare
by Basko Healthcare

NL Gebruiksaanwijzing
DE Anwendungshinweise
ENG Instructions for Use
FR Utilisation et application
IT Usi e applicazioni
ES Uso y aplicación

NL

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ENG

Usage guidelines

Your doctor is the one who prescribed the Clubycare shoe and determines the treatment duration. The daily duration of use prescribed by the physician should be carefully adhered to.

The technician fitting the Splint is responsible for the correct fitting and functioning. In the event of skin irritation, abnormal pain or any other complication you believe to be related to the product, please contact the your doctor and/or technician.

Intended use

Clubycare shoes are used for treating club feet, according to the „Ponseti“ method. The Clubycare shoes are an orthopaedic component. The complete orthotic is comprised of a P.E.A.care splint onto the ends of which the Clubycare shoes are mounted. The orthotic provides the necessary external rotation of the feet and positioning according to the width of the shoulders. This will retain the foot correction achieved during preliminary treatment and help prevent relapse.

Indications:

- Arthrogryposis multiplex congenita
- Pes equinovarus congenitus

Additional product information

The shoes are not suitable for use in water.

Inherent risks and contraindications

Product is **not** suitable for standing or walking. The shoes may not be worn on irritated skin. There are no other known risks for use or contraindications.

Care advice

Check the shoes regularly for wear on the inside, closures and outside.

Material specifications

Shoe: leather; sole: plastic and zinc plated steel

Cleaning instructions

Clean the shoes with a cloth and warm water to a max. 40°C. Do not dry-clean. Not suitable for washing machine. Do not tumble dry.

Instructions

Check the packaging immediately:

- A)** 1 pair Clubycare shoes
- B)** Instructions for use

Donning the shoe

Step 1: Measure the length and the width of the foot and refer to the size chart in order to select the correct size (Fig. 1, see size chart backside).

Step 2: Mount the footplates of the splint to the sole of the Clubycare shoes using the hexagonal cylinder head screws and Allen wrench provided (Fig. 2).

Step 3: Open the shoe's lace/ Velcro fasteners and instep strap (Fig. 3).

Step 4: Position the foot in the shoe. Make sure the heel is positioned properly towards the back of the shoe. Check for correct positioning via the heel cut-out (Fig. 4). It is very important for proper treatment that there is no space between the insole of the shoe and the heel.

Step 5: Place the strap over the instep, pull it to the back and fasten it. Check again that the foot is properly positioned towards the back (Fig. 5). The shoe is provided with two slots for the instep strap to optimize heel placement.

Step 6: Pull the top and bottom of the sock or stocking tight so there are no folds inside the shoe. (Fig. 6).

Step 7: Fasten the lace or velcro fasteners and repeat steps 3 to 6 with the other foot and shoe (Fig. 7).

Step 8: To mount the shoes on the splint, slide the shoe with the foot plate sideways onto the splint. Press and click the foot plate into position onto the adapter. To separate them, press the red release clamps together and slide the foot plate off the adapter (Fig. 8).

Tip: It is possible to customize the instep strap and Velcro straps by cutting them off between the stitching.

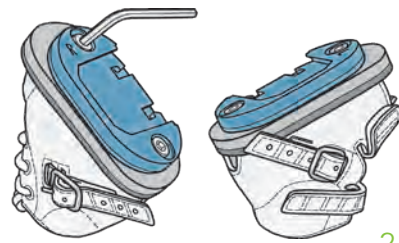
Note:

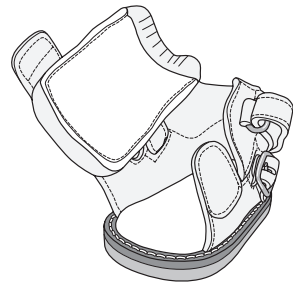
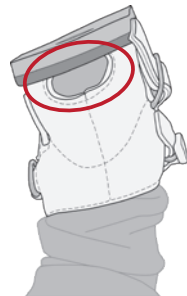
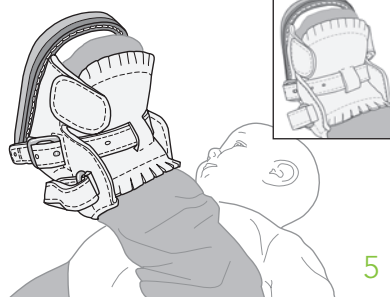
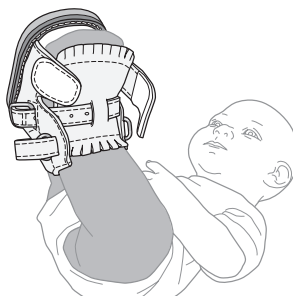
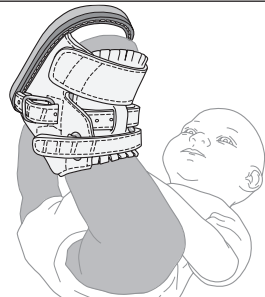
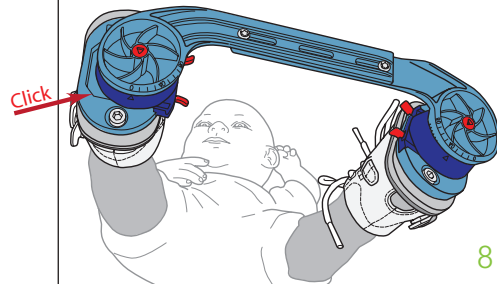
THIS PRODUCT IS **NOT** SUITABLE FOR STANDING OR WALKING.

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